REF

2030005

International Offices:

i3 Membrane GmbH
Theodorstraße 41 P
DE-22761 Hamburg
Germany
P: +49-40-2576748-0
F: +49-40-2576748-48
info@i3membrane.de

North America
i3 Membrane Corporation
13801 Reese Blvd West. Ste 310
Huntersville NC. 28078
USA
P: +1 980 888 81 17
support@i3membrane.com

i3 Membrane GmbH
Rindlberg 103
AT-3972 Bad Großpertholz
Austria
P: +43-676 58 119 57
atbestellungen@i3membrane.de

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DE i3 FIVE - inline filter for secure bacteria retention EN

Intended use
The i3 FIVE inline filter is used for the filtration of tap water. It is installed in the inflow pipes of devices and instruments to prevent contamination due to waterborne bacteria such as *Legionella pneumophila*, *Pseudomonas aeruginosa* and *mycobacteria*. The i3 FIVE filter is used for reliable microfiltration to exclude waterborne pathogens in situations involving high levels of water flow. The bacterial filter is used in settings such as medical device re-processing (e.g. AERs) and the filling of birth pools.

Application
The i3 FIVE inline filter is mounted in the pipe system by means of quick-release connectors. After a 50-day maximum period of use the disposable filter must be dismantled and disposed of. Please read the operating instructions thoroughly before use.

- Safety instructions**
- Installation of the filter and adapters may only be carried out by qualified technical personnel
 - The water supply must be closed when the filter is not in use
 - There must not be any tension (leverage of the connection hoses) on the installed filter
 - Do not install the filter/accessories in tankless (low pressure) systems
 - Do not use the filter outside of its specification
 - The filter can be used for a maximum of 50 days
 - The duration of use may be shorter if water quality is poor (rust, sand, etc.)
 - It is not intended to be used for injection, infusion, irrigation or hemodialysis
 - The filter does not change the chemical properties nor the taste or smell of water
 - Do not remove the filter while it is in use – air in the piping system can block the filter
 - If there are any signs of contamination, then the water filter should be replaced

REF

Article number

Read the instructions thoroughly before use

Do not use after the expiry date

Latex free

SN

Serial number

Do not use if the filter or the packaging is damaged

LOT

LOT number

Manufacturer

Preparation

When installing the filter for the first time, two suitable quick release couplings and optional the wall bracket must be installed (see i3 Couplings). Do not use the filter if the packaging is damaged. Do not use the filter if the use-by date has elapsed (see filter packaging).

- Installation**
1. Disinfect your hands
 2. Perform a surface disinfection of the quick release coupling
 3. Open the packaging of the filter. If possible, do not touch the filter inlet and outlet side
 4. If possible, close the water supply and insert the outlet side of the filter into the corresponding quick release coupling (see arrow on product).
 5. Install the inlet side of the filter on the second quick release coupling
 6. Check whether the filter is firmly installed in the quick release couplings. Open the water supply and check the installation for leaks

- Deinstallation**
7. If possible, close the water supply and disconnect the filter on the inlet side
 8. Then disconnect the filter on the outlet side
 9. Dispose the filter with household trash or according to the local guidelines

Technical data	
Bacterial retention:	> 7 log (<i>Brev. dim.</i>) per cm ² filter area
Flow rate:	20 L/min (5.3 gal/min) at 5 bar (75 psi) initial
Duration of use:	50 days (7 weeks) (Depending on water quality and frequency of use. The actual period of use can be extended up to six months)
Maximum operating pressure:	5 bar (75 psi)
Maximum operating temperature:	≤ 60 °C (140 °F) (max 70 °C (158 °F) for 30 mins)
Storage life:	5 years (at max 40° C - 104° F storage temperature)
Chlorine resistance:	1 hour ≤ 200 ppm active chlorine; 50 days ≤ 10 ppm active chlorine
Disinfection:	Can be disinfected by wiping with common surface disinfectants
Measurements:	40 mm (1.6 in) x 187 mm (73.6 in) (diameter x length)
Adaption:	Quick release coupling

Accessories
G1/2" Quick-release-coupling (OT) 260050 (item no.)
(Other accessories on request)

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G1/2" AG Inline-Snelkoppeling (Art. Nr. 2060050)