Instruction for use
i3 TWO connect
Disposable Water Filter - SHOWER

REF 203200-S

Your Companion
Life Science Filtration

Offices

Europe
i3 Membrane GmbH
Theodorstraße 41 P
DE-22761 Hamburg
Germany

North America
i3 Membrane Corporation
13801 Reese Blvd West. Ste 310
Huntersville NC. 28078
USA

Contact
P: +1 980 888 81 17

support@i3membrane.com
www.i3membrane.com

i3 Membrane GmbH
Christoph-Seydel-Str. 1
DE-01454 Radeberg
Germany

R_x only

Indication for use
The intended use of the i3 Membrane Water Filter i3 TWO connect is to filter EPA quality drinking water. The i3 Membrane Water Filters retain bacteria, which may aid in infection control. The filters produce water that is appropriate for washing and drinking, superficial wound cleansing (minor cuts, scrapes and abrasions), cleaning of equipment used in medical procedures and washing of surgeon's hands.

The i3 Membrane Water Filters are not intended to provide water that can be used as a substitute for USP grade sterile water.

Description
The bacteria retention of the water filter meets the requirements of the ASTM F838 standard.

The filter is provided sterile using irradiation.

The i3 TWO connect has a bacteriostatic additive to prevent retrograde contamination.

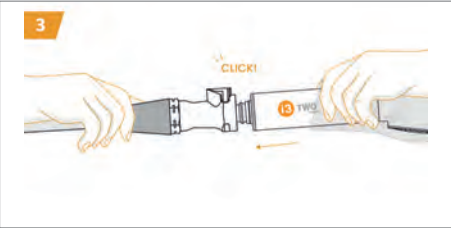
The shelf-life of the filter is 5 years, see expiry date for detailed information.

The filter withstand a temperature of 158 °F (70 °C) for 30 minutes during the whole duration of use of the filter to perform a water system disinfection.

The maximum operating temperature is 140 °F (60 °C).

The product does not contain natural rubber latex.

Follow the instructions provided in this instruction for use.

Preparation
Check the packaging for damage. Only use products with an intact packaging vacuum.
Make sure a correct quick release coupling has been installed on the shower hose.

Installation
1. Disinfect your hands
2. Open the transparent vacuum packaging of the filter on the filter inlet side without touching the filter directly
3. Connect the filter to the quick release coupling with a small turning movement
4. Check the installed filter for leakage

Using
The filter can be disinfected by wiping with the common disinfectant. After initial installation the filter can be used for a maximum of 50 days. Once disconnected, the filter has to be disposed.

Deinstallation
5. Disconnect the filter from the quick release coupling
6. Dispose the filter with household trash or according to the local guidelines if these differ

Contraindications
The filters are not to be used for injection, infusion, irrigation or hemodialysis.

Information
Duration of use:
The duration of use could be reduced due to high particle concentration in the water (rust, sand, etc.).
Tankless systems:
Do not install the filter on tankless (pressureless) systems.
Disinfection by wiping:
The filter can be disinfected by wiping with common disinfectants.
Disposal:
The filter is discarded with household trash.

Precaution
Do not reuse, resterilize or reprocess. If there are any signs of contamination the water filter should be replaced.

The i3 TWO connect is designed for a maximum pressure of 75 psi (5 bar).

Do not use if packaging is damaged.

Technical data
Bacterial retention:
Retrograde protection:

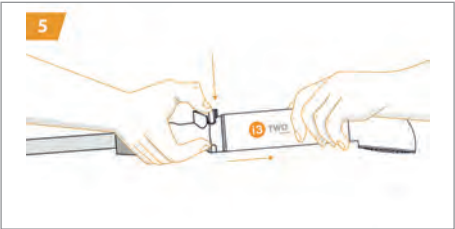
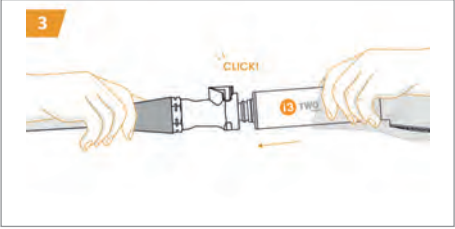
Flow rate:
Duration of use:
Operating pressure:
Operating temperature:
Storage life:
Chlorine resistance:

Material certifications:
Dimensions:
Adaption:

> 7 log (Brev. dim.) per cm² filter area
> 4 log *Pseudomonas aeruginosa*, *Escherichia coli*, *Staphylococcus aureus* per 24 hours
4 gal/min (15l/min) at 75 psi (5 bar) (initial)
50 days (7 weeks)
max. 75 psi (5 bar)
max. 140 °F (60 °C) (max. 158 °F (70 °C) for 30 min)
5 years
200 ppm active chlorine for 1 hours or
10 ppm active chlorine for 50 days
BS 6920:2014
40 x 230 mm (diameter x length)
Quick release coupling;
see www.i3membrane.com

- Symbols**
- Reference number
 - Read the instructions thoroughly before use
 - Do not use after the expiry date
 - Does not contain or presence of natural rubber latex
 - Do not use if the filter or the packaging is damaged
 - Do not reuse
 - Sterilized using irradiation
 - Serial number
 - LOT number
 - Manufacturer

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